

**PRIOR TO ENTERING ROOM**



Ensure that all unnecessary materials are removed from the basket.



Perform precautionary wipe of all surfaces with approved wipe  
**\*\*maintain wet time as stated by manufacturer \*\***  
 For a list of approved wipes and wet times visit [EPA site](#)

**INSIDE THE ROOM, ONCE ULTRASOUND IS COMPLETED, AFTER SANITIZING GLOVES WITH GEL, BUT BEFORE REMOVING PPE**



Visually inspect the machine for any bodily fluid or tissue  
 - If present, remove with wet gauze or wipes



For a list of approved wipes visit [EPA site](#)

Using approved wipe, wet all surfaces of the machine including:

- the keyboard
- the screen
- probes and cords
- the power cord
- the lid
- the wells
- gel bottles
- wipes containers

**\*\*please remember that there is a “wet time” associated with all wipes, check the manufactures recommendation\*\***



While still in PPE, move the wet machine out of the room

**\*\*If another person is unavailable to help, use your foot to push the machine out\*\***

**AFTER EXITING THE ROOM, WHILE WEARING CLEAN PPE**



Visually inspect the machine for any bodily fluid or tissue.  
**If present, remove with wipes**

<https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>

Using an approved wipe, wet all surfaces listed above again

**\*\* Dry completely between the 2 required “wets” \*\***



Maintain wet for **required amount of time** before considering the device decontaminated

**\*\*Please check the wet times of you approved wipes and don't forget to wipe the gel bottles and wipes container as well\*\***